Scarban®
Professional Scar Treatment

INSTRUCTIONS *for use*

SCAR TREATMENT AFTER C-SECTION

Because every scar needs care!

www.scarban.eu

SCAR TREATMENT AFTER C-SECTION

THE BEST TREATMENT FOR YOUR SCAR STARTS NOW!

Surgery like a caesarean section always comes with a scar. A scar that you obviously want to heal properly and reduce as much as possible.

With Scarban® C-Section silicone therapy you will provide your scar with the best possible treatment. Scarban® silicone sheets are the best choice, which is why they are recommended throughout the world.

Scarban® silicone sheets are highly effective and have been clinically proven to reduce scarring and complaints by creating a semi-occlusive layer on the skin.

This layer reduces water loss through the scar, which is one of the most important precursors for abnormal and disfiguring scar formation.

The result:

- Scar formation is reduced / minimised
- Significant relieve of complaints such as itching and pain
- Improved appearance of the scar, less redness

In conclusion, Scarban® C-section provides your skin with optimal support in the prevention of abnormal scarring.

So start your therapy today for a beneficial recovery!

TIPS

TIPS FOR USE

Please note that Scarban® silicone sheets are only intended for use after complete wound closure.

Start Scarban® silicone therapy after stitches are removed and scabs have disappeared.

For best results it's important to wear the Scarban® silicone sheet on the scar for at least 12 consecutive hours, every day.

Apply the Scarban® silicone sheet to clean, non-greasy skin. Also, do not use greasy creams or oil, as this will reduce the sheet's adhesive strength.

Scar recovery takes time. Please note that it can take up to 6 months, or even longer, for a scar to mature and recover optimally.

Clean the Scarban® silicone sheet daily. For example during your regular shower. This increases the sheet adhesive strength and, thus, its useful life.

Always keep your damaged skin out of the sun. Exposure to the sun can cause sunburn and permanent discolouration.

Scarban® silicone sheets are safe and effective at both new as old scars!

INDEX

Page

Introduction	2
Tips	3
Practical uses	4
EN	5
NL	6
FR	7
DE	8
ES	9
PT	10
IT	11
FI	12
PL	13
TR	14
AR	15
CN	16
TW	17
KO	18
JP	19

For further advise and practical usage see the instructions for use in your language in this booklet.

PRACTICAL USES

1

If necessary, the Scarban® silicone sheet can be cut to size. Ensure that the area 2 cm around the scar is covered with the silicone sheet.

2

For good adhesion and durability of the Scarban® silicone sheet it is advisable to remove any hair from the scar site. Clean and dry the skin thoroughly before applying the silicone sheet to the scar and the immediate, surrounding area.

3

Remove the protective film from the back of the Scarban® silicone sheet.

4

Stick the Scarban® silicone sheet to the scar.

5

If necessary, the Scarban® silicone sheet may be fixed in place with an adhesive or elastic bandage. The skin may be treated with a hydrating cream after removal of the silicone sheet.

6

Scarban® silicone sheets should be cleaned daily to maintain adhesion. Rub the silicone sheet with a few drops of Scarban® Silicone Cleaning Soap or a neutral soap.

7

Remove any dirt as much as possible using soap and lukewarm water. Rinse the silicone sheet well under lukewarm, running water.

8

After contact with soap and water the adhesive effect disappears and the Scarban® silicone sheet rolls up. Allow the silicone sheet to dry on a clean, dry surface with the adhesive side facing up. During drying, the silicone sheet automatically unrolls and the adhesive effect gradually returns.

Name:

Scarban® is a range of high-quality products for the treatment of scars.

Product description:

- Scarban® is a proven technology for the treatment of scars (hypertrophic and keloid) and for prevention of abnormal scar development after surgery or injury. The medical silicone sheet is self-adhesive, washable and can be used over a long period of time.
- Scarban® Cleaning Soap is a cleaning agent for the silicone sheets.

Composition:

- Textile layer: polyamide, elastane
- 100% Medical silicone gel

Properties:

Scarban® silicone sheets ensure that scars become flatter, more flexible and thinner - and that complaints (persistent redness, scar pulling, itching, restricted joint movement, discomfort and cosmetic concerns) decrease. The silicone sheet provides a semi-occlusive (covered) environment on the surface of the skin, resulting in increased hydration of the skin (epidermis). Increased hydration ensures quicker recovery of damaged, scarred skin and reduces functional complaints. Scarban® contains UV protection (UPF50).

Indications/applications:

Scarban® silicone sheets are suitable for:

- both old and new scars
- all skin types
- all ages
- treatment and prevention of abnormal scars (hypertrophic and keloid)
- treatment of burn wound scars
- treatment of surgical scars
- use under clothing or compression therapy
- treatment of persistent functional problems caused by scarring (persistent redness, scar pulling, itching, restricted joint movement, discomfort and cosmetic concerns)

Instructions for use:

- For optimal results, preferably start Scarban® silicone therapy immediately after the wound has healed closed.
- It is recommended that Scarban® silicone therapy be built up during the first week to a minimum of 12 and a maximum of 20 hours per day. This build-up serves to gradually accustom the skin to silicone.
- For best results, it is preferable to wear silicone sheets for a minimum of 12 to a maximum of 20 hours per day.
- If necessary, the Scarban® silicone sheet can be cut to size (Fig. 1).
- Ensure that the area 2 cm around the scar is covered with the silicone sheet.
- For good adhesion and durability of the silicone sheet it is advisable to remove any hair from the scar site.
- Clean and dry the skin thoroughly before applying the silicone sheet to the scar and the immediate, surrounding area (Fig. 2).
- Remove the protective film from the back of the silicone sheet (Fig. 3) and stick the Scarban® sheet to the scar (Fig. 4).

- If necessary, the silicone sheet may be fixed in place with an adhesive or elastic bandage (Fig. 5).
- The skin may be treated with a hydrating cream after removal of the silicone sheet.

Maintenance:

- Scarban® silicone sheets should be cleaned daily to maintain adhesion.
- Rub the silicone sheet with a few drops of Scarban® Silicone Cleaning Soap or a neutral soap (Fig. 6).
- Remove any dirt as much as possible using soap and lukewarm water.
- Rinse the silicone sheet well under lukewarm, running water (Fig. 7).
- After contact with soap and water the adhesive effect disappears and the silicone sheet rolls up (Fig. 8).
- Allow the silicone sheet to dry on a clean, dry surface with the adhesive side facing up (Fig. 8).
- During drying, the silicone sheet automatically unrolls and the adhesive effect gradually returns.
- Only use the silicone sheet when it is completely dry.
- The edges of the silicone sheets may be trimmed if the adhesive effect deteriorates as a result of wear.

How long can I use Scarban®?

Use Scarban® for as long as the scar is raised, reddened or causes complaints. Discuss length of treatment with your therapist or doctor.

Duration of use:

Each Scarban® pack contains one or more silicone sheets. With correct use and maintenance (according to the instructions for use), each pack provides sufficient contents for about 6-8 weeks of treatment. Several packs may therefore be necessary for long-term therapy.

Special care and precautions:

- Keep out of the reach of children.
- The silicone sheets are only suitable for external use.
- Only use the silicone sheets on closed skin (after the scab has disappeared) and never on open wounds.
- Never dry the silicone sheet in the sun or near a heat source.
- Temporarily remove the silicone sheet while exercising or swimming. Scarban® may detach itself due to perspiration or contact with water.

Side effects:

In rare cases the skin may become hypersensitive (skin irritation) to silicone treatment. If so, stop the treatment until the skin has settled down. Start the treatment again and allow the skin to become accustomed to the silicone sheet more slowly (one hour longer every day). If skin problems persist, consult your therapist or doctor.

Storage:

- See pack for expiry date.
- Store at temperatures between 5 and 25 °C.

Date of last revision of instructions for use: April 2020

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